

Set-up Instructions



1) Preparing Your Anutra Dispenser®

Open the Back Door on your Anutra Dispenser® by gently squeezing the ribbed notches at the top of the Back Door and pull backwards.

Now, insert your loaded Anutra Cassette® into your Anutra Dispenser®. See the label on the top of your Cassette (Ref: 02) for vial loading instructions.

Once the Anutra Cassette® is fully inserted, replace the Back Door of the Dispenser.

* When using a single vial, where nothing is attached to the luer lock adapter, the gray knob and red slide is not used.



2) Activating Your Anutra Cassette®

Attach any luer lock syringe to the SmartSite® and twist clockwise until the syringe is securely locked.

Pull backwards on the syringe plunger rod until fluid begins to flow. This activates the fluid flow through the Cassette. Discard contents of this syringe.

3) Dispensing

To dispense fluids, wipe the SmartSite® with an alcohol swab before each use. Attach any luer lock syringe to the SmartSite® and twist clockwise until the syringe is securely locked. Pull backwards on the syringe plunger rod until you have the desired amount of fluid in your syringe.



4) Unloading the Anutra Dispenser®

Remove the Back Door of your Anutra Dispenser®. Pull backwards on the Anutra Cassette® to remove from the Anutra Dispenser®.

Discard the Anutra Cassette®. Never remove vials from an Anutra Cassette®. The Anutra Cassette® is designed for single-use only.

Caution:

- ✦ Once you have used a luer lock syringe on a patient, never reattach to the Anutra Cassette®.
- Do not push in the plunger rod on the syringe while attached to the SmartSite®, especially after drug has been drawn into the syringe.
- Always attach a luer lock syringe to the SmartSite **before** dispensing fluids.
- Discard the Anutra Cassette® fourteen days after attaching the drugs or after the vial is empty. Never detach drugs from cassette once attached.
- Refer to the drug manufacturer's insert for dosage, contraindications and adverse reactions.

Indications for use: Refer to the drug manufacturers instructions for use.

For additional training visit: www.AnutraMedical.com/training